



# NuEra Tight

## A New Era of Personalized Body Sculpting Treatments

Radiofrequency (RF) has become increasingly popular in recent years as patients desire to sculpt problem areas comfortably and without downtime. NuEra Tight with FocalRF technology uses 5 different frequencies to dial-in the depth of energy penetration heating specific skin and tissue layers achieving Fat Reduction, Skin Tightening, Cellulite and Wrinkle Reduction, all in one device.



Fat Reduction



Skin Tightening



Cellulite Reduction



Wrinkle Reduction



NuAPIC

### Automatic Personalized Temperature Control

Ensuring therapeutic temperature throughout the procedure. Adapting the power to meet the patient's unique needs.



NuLogic

### Advanced Protocol for Expert Users

Multiple treatment protocols applied automatically in one treatment session: frequency, temperature, time and electrode size.




Easy to Use

Advanced user experience with automatic accessory recognition simplifies and speeds up the treatment while maintaining comfort and safety.



# Body Sculpting Takes on a New Meaning

NuEra Tight provides you with advanced tools for today's aesthetic patient.

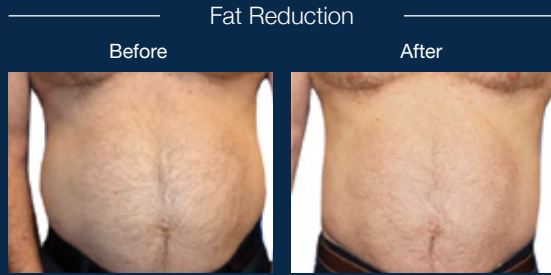


Courtesy of Jeffrey Caruth, M.D.



Courtesy of Clark Schierle, M.D., Ph.D, FACS



Courtesy of Luigi Rizzi, M.D.

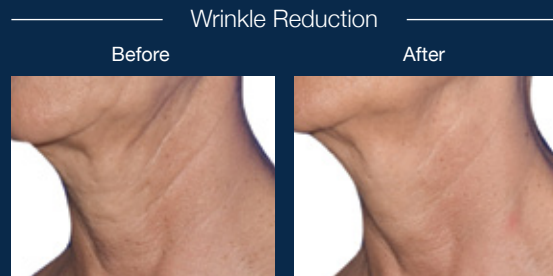

Courtesy of Gerhard Sattler M.D.

## Specifications

### NuEra Tight 1CH is intended for:

- Treatment of edematous-fibrous-sclerous dermopanniculosis
- Treatment of skin atrophy and elastosis
- Treatment of microcirculation and venous insufficiency
- Treatment of hyperplasia and hypertrophy of adipocytes

**Warnings and Risks:** NuEra Tight is unsuitable for anyone who is pregnant, has an infection or illness, or is heat-sensitive. Treatment over cuts, wounds, piercings and tattoos must be avoided and areas with injectable fillers and toxins left for a month before treatment. Patients with a metal implant or implanted device (e.g. pacemaker) must consult with a qualified physician first. If deemed suitable, the return pad must be placed far away from the implant. Local, oral or systemic anesthetics cannot be used before or during treatment as patients need to provide regular feedback to ensure their continued comfort. Insulate patients from metal objects, especially any in direct contact (e.g. the bed). A full practitioner consultation is always advisable; for contraindications and risks in full, see the NuEra Tight user manual.

FocalRF Emission Frequency ( $\pm 1\%$ )	470 kHz, 1 MHz, 2 MHz, 4 MHz, 6 MHz
Output Energy ( $\pm 10\%$ )	250 W
Display LCD Touch Screen	15"
Coupling System	Capacitive
Operative Mode	Automatic treatment presets, NuLogic & Manual
Temperature Control	NuAPIC
Temperature Range	From 35°C to 45°C $\pm 1^\circ\text{C}$
Timer	Up to 90 min
Mains	100-240 Vac; 50/60 Hz; 350 W
Dimensions	127x55x60 cm (HxWxD)
Weight	Weight 40 Kg
Safety Class	Class I - type BF
Handpieces	Face and Body, Bipolar
Return Electrodes	Disposable return pad
Capacitive Coated Electrodes	$\varnothing 20$ mm, $\varnothing 30$ mm, $\varnothing 40$ mm, $\varnothing 60$ mm, $\varnothing 70$ mm, $\varnothing 80$ mm, $\varnothing 100$ mm, $\varnothing 40$ mm with massage

**Bios S.r.l**  
Via Guido Rossa  
10/12 - 20090  
Vimodrone (MI) - Italy  
T +39 02 27304275  
F +39 02 27304276  
info@biosgroup.eu  
www.biosgroup.eu

**AMERICAS**  
San Jose, CA, USA  
T +1 408 764 3000  
+1 877 586 3647  
F +1 408 764 3999

**AUSTRALIA**  
Chatswood, NSW,  
Australia  
T +61 2 8459 9678

**ASIA PACIFIC**  
Beijing, China  
T +86 10 5737 6677  
F +86 10 5737 6767  
Gurgaon, India  
T +91 124 422 07 95  
Kowloon, Hong Kong  
T +852 217 428 00  
F +852 272 251 51

**EMEA**  
Dreieich Dreieichenhain,  
Germany  
T +49 6103 8335 0  
F +49 6103 8335 300  
Roma, Italy  
T +39 06 90 75 230  
F +39 06 90 75 269

Hertfordshire, UK  
T +44 20 8736 4110  
F +44 20 8736 4119

© LUMENIS LTD 2020

CE<sub>1936</sub> Rx ONLY

**Lumenis**<sup>®</sup>  
Energy to Healthcare



LUMENIS.COM/NUERA-TIGHT

PB-00018870 Rev A